



southpaw enterprises, inc.®

Instruction Sheet

Biorbital Accelerator 1700

Store these instructions in a safe place or with the enclosed maintenance checklist




Take time to familiarize yourself with the use and maintenance of this equipment before using. Please file this Instruction Sheet in an accessible location: the Instruction Sheet must be read and familiar to anyone who is using Southpaw Enterprises Equipment.

IMPORTANT: Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact Southpaw Enterprises, Inc. immediately.

Warning: Failure to properly install a forged steel eyebolt (1/2" or 5/8") may result in serious injury or death. If you have any questions about the installation of the forged steel eyebolt, please contact Southpaw Enterprises Customer Service at 1-800-228-1698.

MATERIALS LIST

- 1 - Padded Biorbital Accelerator Bar
- 1 - Set of Leg Loops and Chest Harness
- 1 - Set of Leg Loops with Safety Snap
- 1 - Sling Seat and Chest Harness
- 1 - Safety Rotational Device
- 4 - Safety Snaps
- 1 - Instruction Sheet
- 1 - Safety Snap Instruction Sheet
- 1 - Safety Rotational Device Instruction Sheet
- 1 - Maintenance Checklist

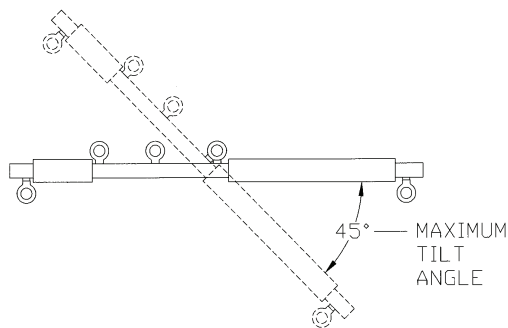

MINIMUM HEIGHT REQUIREMENT : 8 FT.

Use care to assure a clear distance of 6 ft. from walls and other obstacles. A Southpaw 5000 Height Adjuster is recommended for ease in adjusting the height of your equipment. The Safety Rotational Device included with this equipment must be used at all times.

WORKING LOAD: 300 lbs.

MAXIMUM TILT ANGLE:

Do not exceed the 45 degree tilt angle as shown above. The eyebolts are designed for maximum strength at a straight pull. They are not designed for a right angle or sideways pull. Failure to observe this warning could result in the eyebolt breaking, causing serious injury to users.

PROCEDURE FOR HANGING:

- To hang your Biorbital Accelerator hook the safety snap that is attached to the Southpaw Safety Rotational Device to your suspension system.
- The main pivot point of the steel crossbar can be moved by shifting the Safety Rotational Device to one of the three positioning eye bolts to help balance the weight. For clients of equal weight use the bolt at the center of the bar. To balance the weight for clients who do not weigh the same, move the Safety Rotational Device toward the heavier client. This must be done with the clients off the equipment.
- The Biorbital Accelerator can be used with either one set of leg loops and one Sling Seat or with two sets of Leg Loops.
- The chest harness must be used on clients at all times. Do not use Biorbital Accelerator with bar padding removed.

MAINTENANCE CHECKLIST

Biorbital Accelerator must be checked before each use. Please refer to Safety Snap and Safety Rotational Device Instruction Sheets for further instructions on these items.

- Do not remove plastic end caps to attempt repair of built in rotational devices.
- Do not grease or lubricate rotational in any way. If rotationals do not spin freely, contact Southpaw.
- Check the Safety Rotational Device and the two built in rotationals on the ends to make sure they spin freely. If not, discontinue use immediately.
- Check for noise coming from rotational. If found, discontinue use.
- Check chest harness buckle to make sure it is not broken and is functioning properly. If buckle is broken or not functioning properly, discontinue use immediately.
- Check for fraying of ropes. If excessive, discontinue use until repaired. Check ropes for splice locks* at every point that the rope is "looped" into itself. If any are missing, discontinue use until repaired.
- Check web loops and seat for loose or missing stitches. If stitches are loose or missing, discontinue use until repaired.

* Splice Locks are the rope staples.

MATTING PRECAUTIONS: Always use appropriate matting

CLEANING INSTRUCTIONS

- Check with your infectious disease department before using these procedures.
- Check for client allergies first.
- Clean with a damp cloth using soap and water or disinfectant.

STORAGE PROCEDURES

- Store out of direct sunlight.
- Store where not accessible by clients without proper supervision.

CAUTION

- ♦ **Check ceiling eye bolt for wear and movement. Use a Southpaw Safety Rotational Device to prevent abnormal wear.**
- ♦ **Floor mats of proper thickness should be positioned properly under and around all suspended equipment.**

Safety Precautions - The safety of your clients is dependent on the appropriate choice of activity to match, yet challenge their physical abilities. Use appropriate spotting / safety techniques. In addition, the layout of your activity area, proper installation and maintenance of your equipment is critical. Southpaw equipment is not intended for recreational or outdoor use. Under no condition should this equipment be used without proper supervision.

This sheet is for example only. It is not meant to, in any way, replace local, county, state, or Federal standards. Be aware - it is your responsibility to find all pertinent safety information that pertains to your facility.



If you have any questions or comments, please call or write
Southpaw Enterprises, Inc.
P.O. Box 1047, Dayton, Ohio 45401 -1047
Toll Free - (800) 228-1698
Come see us on the World Wide Web at <http://southpawenterprises.com>