

## Operation

- 1. Open the system kit box.
- 2. Charge the PlayBall for at least  $\sim 1$  hour.
- 3. Turn on the Control panel (Make sure it is charged).
- 4. Load PLAYWORK SW app.
- The connection is automatically. Wait until the software is connected to the PlayBall - until you will see round green light on the upper-right side of the screen.
- 6. You are ready to go. (If you are facing problems please contact our <u>customersupport@playwork.me</u> to fix it before the session).

# Description


### PlayBall: The Smart Therapy Exercise Ball

PlayBall is an intelligent Rehab & fitness ball used with the PLAYWORK interactive recovery program.

PlayBall is a breakthrough smart exercise ball functioning as a performance measuring tool and video game controller that allows you to complete rehabilitation ball exercises whilst playing games and receiving real-time visual feedback. Explore engaging games designed with therapists to enhance complete body workouts.

PlayBall smart integrated sensors measure and track your performance (e.g force, motion and more) allowing you to monitor your progress and manage recovery programs.

The only interactive physical therapy equipment that measures both movement and pressure (force applied on the ball).

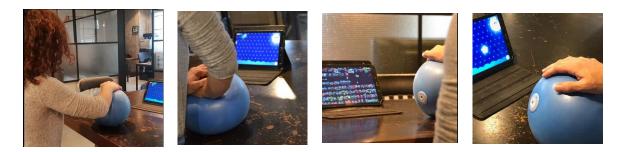



### Total body exercises


Play games on the PlayBall to complete full body rehabilitation exercises. PlayBall includes a unique game mechanism that offers you a list of recommended ball workouts and also allows you to work "Freestyle" by inventing your own ball exercises.





### Engaging games

Discover a unique engaging game package and set new high scores. All games are personalized and set automatically according to your personal parameters.

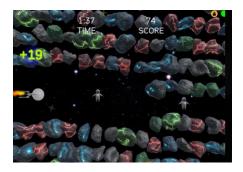

Space Advanture Press and release mechanism to help the spaceship fly through the asteroids.



Flying Rocket Pressing in short intervals will keep the space rocket on the right track in various missions.



Monster Blast Roll the ball to blast as many monsters you can while avoiding bad enemies. Applying force whilst rolling the ball will help you do even better!



Bottle vs. Bullet If you have a crazy for shooting games, get ready to take up the challenge of shooting bottles at this eye hand coordination activity



Knife Hit Apply force in the right timing to cut as many apples you can in a "fruit ninja" type challenge




## "Performance" workouts and dashboard

PlayBall's smart algorithms measure and track your performance to create an even more engaging and efficient recovery process.



#### Dashboard

Every workout is logged and reflected in mission to provide the most successful and efficient recovery process. Real-time measures and stopwatch feature allows you to conduct "performance" exercises without a game.

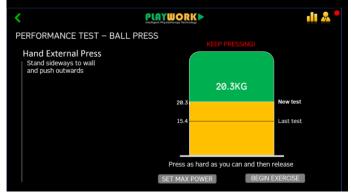

#### **Performance Test**

From time to time you will be asked to test your max power. Results will automatically reset your personal parameters and game controls.

### Specifications

Color: Blue, Yellow, Green.

Dimensions: 17-20cm (diameter) fitness ball. Soon will be available in various ball dimensions. Control panel : 10.1' touchscreen, Quad core processor 1.3GHz, 1GB DDR3 RAM, 16GB internal storage.



## **PlayBall Basic**

Smart PlayBall kit includes smart ball with motion & pressure sensors. SW Lic. for #50 users including software upgrades and additional games. 10'' control panel Basic warranty for first year with partial coverage: online service and support. Repair and replacement with additional fee.

# **PlayBall Premium**

Smart PlayBall kit includes smart ball with motion & pressure sensors. SW Lic. for unlimited users including software upgrades and additional games. 10'' control panel Extended warranty for first 2 years with full coverage: onsite service and support. Repair and replacement with no additional fee.

# **My PlayBall**

Smart PlayBall kit includes smart ball with motion & pressure sensors. SW Lic. for #1 user including software upgrades and additional games. 7'' control panel Basic warranty for first year with partial coverage: online service

and support. Repair and replacement with additional fee.







#### PLAYWORK Ltd.

Yasha Hefetz St. 11 Tel Aviv, Israel Tel: +972-50-9300900 | info@playwork.me